Cat. No 301

GET READY!

babyono


<b>GB</b>	ANATOMY Manual breast pump	2
<b>PL</b>	Laktator ręczny ANATOMY	4
<b>BG</b>	Ручний молоковідсмоктувач ANATOMY	6
<b>CZ</b>	Ruční odsávačka ANATOMY	8
<b>DE</b>	Manuelle Milchpumpe ANATOMY	10
<b>ES</b>	Extractor de leche manual ANATOMY	12
<b>FR</b>	Tire – lait manuelger ANATOMY	14
<b>GE</b>	მკერდის მექანიკური ტუმბო ANATOMY	16
<b>GR</b>	Συσκευή άντλησης μητρικού ANATOMY	18
<b>HU</b>	ANATOMY Kézi mellszívó	20
<b>LT</b>	Rankinis pientraukis ANATOMY	22
<b>LV</b>	Rokas tipa piena pumpis ANATOMY	24
<b>NL</b>	Handkolf ANATOMY	26
<b>PT</b>	Bomba Tira Leite Manual ANATOMY	28
<b>RO</b>	Pompa de sân manuală ANATOMY	30
<b>RU</b>	Ручной молокоотсос ANATOMY	32
<b>SE</b>	Manuell bröstpump ANATOMY	34
<b>SK</b>	Ručná odsávačka mlieka ANATOMY	36
<b>TR</b>	ANATOMY El göğüs pompası	38
<b>UA</b>	ручний молоковідсмоктувач ANATOMY	40
<b>XS</b>	Ručna pumpica za izmuzavanje majčinog mleka ANATOMY	42

**WARNING!**

Read carefully the instructions for use and warnings before the first use of the Babyono breast pump. Keep the instructions for future reference!

**BREAST PUMP COMPONENTS**

1. Silicone massaging pad
2. Breast pump funnel
3. Pump cover
4. Silicone check valve
5. Bottle
6. Main body cover
7. Silicone main body seal
8. Suction roller
9. Main body
10. Handle
11. Teat
12. Cap with a ring
13. Sealing bottle cap

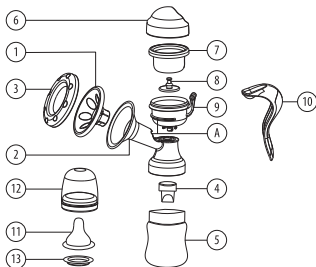**INSTRUCTIONS FOR ASSEMBLY**

1. Place the massaging pad (1) on the breast pump funnel (2) and press it to form a whole.
2. Combine the suction roller (8) with the silicone seal (7) from the bottom.
3. Place the breast pump main body (9) on its funnel element (2) and give it a twist to the
4. moment where the points marked in figure with A letter are aligned.
5. Next put the silicone seal (7) on the breast pump main body (9). Place the handle (10) on the suction roller (8), fasten to the main body (9) of the device and put the main body cover (6).
6. Attach the silicone check valve (4) to the funnel (2), then connect the bottle (5) to the breast pump funnel.

**INSTRUCTIONS FOR USE**

1. Remember that stress has a considerable influence and impact on the amount of produced milk. Before you start expressing milk make sure to eliminate any situations stressful to you. Try to relax and always remember about hygiene: wash your hands with soap and water and your breast with water only.
2. Many women feel more thirsty during breastfeeding or expressing of milk – prepare a glass of water or anything else to drink in order not to stop expressing milk.
3. Do not be discouraged with the initial failures – you will shortly become skilled and expressing of milk will not pose a problem to you.
4. Sit down in a position comfortable to you that will enable you to lean slightly forward.
5. Take the previously disinfected breast pump (make sure that all the elements are correctly assembled). Place the breast inside the pump main body adjusting it to the silicone pad in order to place a nipple deep enough to provide its proper stimulation by petals on the pad.
6. An especially designed construction of the handle lever allows setting it at an angle suitable for the mother by positioning it to the left or right hand.

7. By pushing down the pump handle with rapid movements massage the breast and stimulate it to lactation. Next rhythmically push down the handle and release the pressure making milk flow into the bottle. Adjust the rhythm and pressure to your own needs.
8. If you do not feel any suction check whether there is any air between the breast and the silicone pad. If there is some air there, take out the breast and place it again in the pump funnel.
9. Stop expressing your milk when it ceases flowing from the breast.
10. The amount of expressed milk will vary; it depends on whether the lactation process is regulated, on a time of the day and amount of liquids that you drink. Expressing of milk from full breasts may take even 30 minutes.
11. The use of the breast pump should not be painful. If the process causes pain to you, if you can see redness and irritation of the breast, immediately consult a breastfeeding advisor.
12. Following the completion of expressing of milk wash all the pump elements and prepare them for sterilisation.

## ASSISTANCE OF THE BREAST PUMP IN BREASTFEEDING-RELATED PROBLEMS

- Excess of milk – when the breast is filled with too much milk it is difficult for the baby to correctly grasp a nipple. By expressing the excess of milk you will make the breast softer and thus you will make it easier for the baby to correctly grasp the nipple.
- Shortage of milk – insufficient amount of milk makes the baby more easily irritated and discourages it from sucking the breast. By using the BabyOno breast pump between proper breastfeeding you will stimulate your breast to produce more milk.
- Nipple shape – especially in the initial phase of breastfeeding numerous women complain about the shape of their nipples that are not sufficiently protruding which makes it difficult for the baby to correctly grasp the nipple. Regular use of the breast pump will help you to more quickly adapt the breast for feeding.
- Sore nipples – before you and your baby become skilled in breastfeeding the nipples may be wounded with gums of the infant who may suck them in an improper way. By using the BabyOno breast pump you give your nipples some time for healing and shortly you will be able to breastfeed the baby again.

## CLEANING OF THE BREAST PUMP

Disassemble all the breast pump parts (in an order opposite to the assembly instructions). Before the first use and after each subsequent use carefully clean all the pump elements in warm water with soap for children and rinse several times. The breast pump may be also washed in the dishwasher (on the top rack only), but you must remember to immediately take out and dry all its elements following the completion of the program.

After washing the breast pump and each time before its use all the part having contact with milk should be sterilised:

- silicone massaging pad (1)
- breast pump funnel (2)
- silicone check valve (4)
- silicone main body seal (7)
- suction roller (8)
- main body (9)
- bottle (5), teat (11), cap with a ring (12), sealing cap (13)

In order to do so boil them in water for 5 minutes or sterilise in a microwave or steam steriliser intended for that purpose. Never use for sterilisation any strong chemicals or any other products not intended for that purpose. Do not leave the product in disinfectants for time longer than the recommended one since it may result in its damage and/or impairment.

Store the product in a dry place at room temperature. Do not leave the product in a sun filled or hot place.

The attached teat is suitable for standard BabyOno bottles. Before each use always check by pulling into different directions whether the teat is not damaged, especially when the child already has teeth. Replace a damaged, bitten or misshaped teat with a new one! It is not allowed to make holes in the teat bigger due to the risk of damage to the product and/or child's choking. Due to hygienic reasons and to ensure safety it is recommended to replace a teat every 1-2 months.

## FOR THE SAFETY AND HEALTH OF YOUR CHILD!

### WARNING!

- Always use the product under the supervision of adults. Never leave the child unattended during feeding.
- Never use bottle teats as soothers.
- The continuous and prolonged sucking of liquids may result in dental caries.
- Always check the food temperature before feeding.
- Keep all the breast pump parts away from reach of children.
- Remember to assemble the sterilised parts of the breast pump and bottle with clean hands.

Product compliant with the EN 14350 standard

### STORING EXPRESSED MILK

**IMPORTANT:** remember to properly store expressed milk! Improperly stored milk loses its nutritive value and may even do harm to the baby.

We recommend sterile Babyono bags for storing and freezing breast milk.

Storage place	Storage time	
	For healthy babies	For preemies and sick babies
At room temperature	8-12 hours	0-1-4 hours
In the fridge (-5°C)	3-4 days	24-48 hours
In the fridge freezer compartment (-10°C)	2 weeks	1 week
In the freezer (-18°C)	6-12 months	3-6 months

### NOTES

- Never refreeze breast milk. Any unused milk should be discarded.
- Do not mix defrosted breast milk with fresh breast milk.
- Only fresh breast milk should be frozen.
- It is recommended to slowly defrost breast milk. For that purpose frozen breast milk should be left in the fridge for reaching a liquid form.
- For making breast milk warm in it recommended to use hot water bath the temperature of which should not exceed 50°C.
- In order to preserve the nutritive value breast milk should not be boiled, warmed up directly on a gas burner or in a microwave!
- The temperature of the milk ready for serving should be the temperature of human body, so not higher than 37°C.

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